GoutChew Duckface ®

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GoutChew Duckface ®

What is GoutChew Duckface ® and when should it be used ?

GoutChew Duckface ® is an orthodontic oral orthosis. Its principle is based on the functional rehabilitation of tongue positioning.

It appears that an orthodontic treatment is only stable if the causes have been eliminated, the origin of which, in almost all cases, is functional and not hereditary. The bad position of the teeth is only the result of an imbalance between the muscular forces of the tongue, cheeks and lips. This inadequate activity of the perioral musculature manifests itself during swallowing, chewing, speaking, breathing, but even more so during the resting position of the tongue. The tongue is composed of 17 muscles, and depending on the degree of contraction of each, it can take very different forms. It is the position of the tongue and the lips that must ensure a harmonious development allowing an easy placement of all the permanent teeth.

GoutChew Duckface ® has the effect of moving the upper lip away from the lower lip, forcing the tongue to spontaneously discover a new dynamic in order to achieve a normal position.

GoutChew Penguin ® should be used in case of:

- Persistence of infantile swallowing in children, adolescents or adults.

When should GoutChew Duckface ® not be used ?

GoutChew Duckface ® is not suitable for children under the age of two.

What precautions should be taken when using GoutChew Duckface ® ?

The device and its packaging box must be handled by an adult.

How to use GoutChew Duckface ® ?

Wear it for 15 minutes a day in a sitting or standing position and/or during a recreational activity (game console, computer, television, ...) for a maximum of one month. If this first phase is insufficient, a slight variation to this daily exercise will be prescribed. In case of prolonged use, the manufacturer does not guarantee the maintenance of the device's performance.

First, position the upper lip in its groove. Second, place the bottom lip in its groove (marked with the Duckface). If you have problems placing the " Duckface ", contact your practitioner who will explain the correct procedure again. It is indeed important to use GoutChew Duckface ® in the required conditions.

What side effects can GoutChew Duckface ® cause ?

There are no known side effects.

What else should I watch out for ?

1. Maintenance

Before the first use, and after each use, you must clean ' Duckface '' with a soft brush, soap and warm water and keep it in its box. Never use chlorinated products such as bleach or products containing peracetic acid.

2. Breathing

When wearing GoutChew Duckface ®, you must always breathe through your nose, but it is exceptionally possible to breathe through your mouth thanks to the easy opening between the two mouthpieces.

3. Use

This therapeutic tool is for individual use only and can be used for the duration of the treatment. It must be disposed of at the end of the treatment.

4. Advices

Gradually offer children harder food, such as raw vegetables to nibble or certain fruits to stimulate their chewing system and optimal development.

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