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ENG

WHAT IS GOUTCHEW RELAX HOME® AND WHEN SHOULD IT BE USED ?

GoutChew Relax home® is a bioactive oral orthosis with an ortho-postural aim. Its principle is based on two actions: proprioceptive stimulation of the perioral soft tissues and the bones of the mouth, and the promotion of balanced muscular work of the jaws, which has an impact on the body posture.

It seems that food in general has become much softer over the decades (cooking, fast food, mixed dishes), requiring almost no chewing effort. The lack of muscular effort and application of the teeth tends to underdevelop the jaw apparatus. Because the bones of the mouth are too small, the space available for the teeth to position themselves correctly in the dental arches is limited. GoutChew Relax home® acts directly on the muscular function of the masticatory apparatus of the adolescent by stimulating the correct growth of the mouth and correcting the position of the permanent teeth in the dental arches (also in adults).

It also seems that the jaw joint (temporomandibular joint) is a sensor that allows us to position ourselves in space (see postural system) as important as the eyes or the feet. This joint allows our jaw to move in all directions in space, which makes it the most complex joint in the human body. An imbalance on one side of this double joint can cause the jaw to be out of position and compensated for by the body, resulting in an alteration of the body's overall posture and causing ongoing tension. GoutChew Relax home® relaxes and rebalances the forces acting on the temporomandibular joints, relieving the body of any compensatory tension that overloads it, allowing it to recover and rebound.

GOUTCHEW RELAX HOME® SHOULD BE USED WHEN YOU :

- Suffer from unexplained migraines;
- Suffer from neck or back tension;
- Suffer from unexplained body pain;
- Grind your teeth;
- Feel anxious or stressed;
- Have a tendency to breathe through the mouth;
- Lack space for front teeth (teeth that overlap and protrude);
- Nighttime snoring or sleep apnea;
- Have a disorder of body posture;
- Have pain related to tooth eruption.

WHEN SHOULD GOUTCHEW RELAX HOME® NOT BE USED ?

GoutChew Relax home® is not suitable for people suffering from epilepsy or convulsions. The use of the oral orthosis is also contraindicated :

- If your teeth are very mobile: If you feel your permanent teeth moving when you apply manual pressure, you should not use the brace;
- If you have an incisor that is significantly longer than the others: difference equal to or greater than 3mm;
- For 3 months following the end of an orthodontic treatment;
- What precautions should be taken when using GoutChew Relax home® ?

• MAINTAIN OPTIMAL ORAL HYGIENE

Brushing your teeth should never be omitted, especially before bedtime, as wearing this device requires you to reinforce your oral hygiene.

• RESPECT THE ADAPTATION TIME

Wearing the bioactive orthosis requires a period of adaptation that can vary from one person to another. Therefore, it is recommended to wear it gradually, starting with a few minutes a day, up to half an hour. If active use causes discomfort in the jaws, it is advisable to use it passively (without exercise) until the situation returns to normal.

• ADJUST THE GUTTER

If you experience nausea or the brace is too large, it is advisable to gradually shorten the side arms until there is no more discomfort.

HOW TO USE GOUTCHEW RELAX HOME® ?

It is advisable to clean the device before use. Wear the bioactive mouthpiece progressively for 5-30 minutes a day when walking or doing sports. For intense physical efforts requiring mouth breathing, it is advisable to use it just during the warm-up. Perform daily exercises by chewing the mouthpiece with your lips still together. Do not bite the mouthpiece (squeezing does not provide any beneficial effect) but chew it in order to benefit from a greater muscular relaxation, it is advised to do small liberating exercises after a few minutes of use:

- Turn and tilt the head to each side;
- Turn and tilt the trunk to each side;
- Lean forward, legs straight, hands as close as possible to the ground;
- Squat down and stand up slowly, rolling out the back;
- Repeat these exercises 2-3 times.

After 7 to 10 days, you can introduce nighttime wear to maximize the benefits, especially if you grind your teeth (bruxism), suffer from sleep apnea, are under stress or if your snoring is annoying.

WHAT SIDE EFFECTS CAN GOUTCHEW RELAX HOME® CAUSE ?

It is possible that the incisors are sensitive at the beginning of the treatment but this disappears quickly. Hypersalivation is also normal. Salivation will gradually decrease and you will be able to wear the device continuously. Don't worry if you can't keep it in your mouth all night, especially if you breathe through your mouth a lot. The key is repetition and regularity of the daily exercises to obtain the benefits.

In case of irritation of the palate and hyper-sensitivity on the teeth, it is advisable to reduce the wearing time. In the case of a lesion on the palate, it is preferable to stop wearing the brace until it has completely healed.

In the case of nausea and vomiting, it is possible that the posterior branches of the orthosis are too long. In this case you can shorten them with scissors. If, despite everything, the nausea and vomiting persist, it is likely that you are hyper-sensitive of the palate. Please refer to your speech therapist to see if desensitization is possible.

It can happen that the gums bleed at the beginning of the wearing of the mouthpiece. In this case, gradual wear is recommended to ensure that the tissues are strengthened and that the bleeding stops spontaneously. The bioactive brace should not cause any injury or pain. If it does, ask your dental practitioner to adjust the appliance to reduce friction against the mucosa.

WHAT ELSE SHOULD I LOOK OUT FOR ?

1. MAINTENANCE

The bioactive oral orthosis should be rinsed with warm water after each use, brushed with toothpaste once a day. When not in use, it should be stored in its box with the lid open within the first few hours of use to allow it to dry.

2. WEAR

This therapeutic tool is made of silicone, which loses its elasticity and effectiveness over time. The lifespan of the orthosis is 3 to 6 months (depending on factors such as activity level, masticatory strength, maintenance). As the benefits gradually diminish, it is advisable to replace the appliance after 6 months at the latest.

3. DENTAL TREATMENTS

If you are undergoing implant, prosthetic or orthodontic treatment or suffer from periodontitis (loosening of the teeth), limit active wearing of the brace to a maximum of 10 minutes during the day (can still be worn at night) and ask your dentist for advice before using the brace.

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