GoutChew Relax ®

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GoutChew Relax ®

What is GoutChew Relax ® and when should it be used ?

GoutChew Relax ® is a bioactive oral orthosis for ortho-postural use. Its principle is based on the promotion of a balanced muscular work of the jaws, which has a cascading effect on the posture.

It seems that the temporomandibular joint (TMJ) is a sensor that allows us to position ourselves in space (see postural system), as important as our eyes or our feet. This joint allows our lower jaw to move in all directions in space, making it the most complex joint in the human body. An imbalance on one side of this double joint can cause the lower jaw to misalign and the body compensates, resulting in a disruption of overall posture.

GoutChew Relax ® relaxes and balances the forces acting on the temporomandibular joints, relieving them of the compensatory stress that overtaxes them, allowing them to recover and heal.

GoutChew Relax ® should be used when :

- You suffer from unexplained migraines;
- You suffer from tension in the neck or back;
- You suffer from unexplained body pains;
- You grind your teeth;
- Feeling anxious or stressed;
- Have a tendency to breathe through your mouth;
- Have a disorder of body posture.

When should GoutChew Relax ® not be used ?

GoutChew Relax ® is not suitable for people suffering from epilepsy or seizures. It is also contraindicated for the use of oral orthoses :

- If your teeth are very mobile: If you feel that your permanent teeth are moving under manual pressure, you should not wear the splint;
- If you have an incisor that is significantly longer than the others: difference equal to or greater than 3mm; ;

What precautions should be taken when using GoutChew Relax® ?

3. Maintain optimal oral hygiene

Brushing of the teeth should never be omitted, as the use of this appliance requires increased oral hygiene.

4. Respect the adaptation time

Wearing the bioactive mouthpiece requires an adaptation period that can vary from one person to another. It is therefore advisable to wear it gradually, starting with a few minutes a day, up to half an hour. If active use causes discomfort in the jaws, it is advisable to use it passively (without exercise) until the situation returns to normal.

How to use GoutChew Relax ® ?

It is advisable to clean the device before using it. Wear the bioactive mouthpiece gradually for 5 to 30 minutes a day when walking or playing sports. In case of intense physical effort requiring mouth breathing, it is advisable to use it only during the warm-up. Perform daily exercises by chewing the mouthpiece with your lips still together. Do not bite the device (squeezing it does not help), but chew it. To achieve greater muscle relaxation, it is advisable to perform small relaxation exercises after a few minutes of use :

- Rotate and tilt the head to either side;
- Rotate and tilt the trunk to each side;
- Lean forward with legs extended and hands as close to the ground as possible;
- Squat down and slowly stand up, stretching the back;
- Repeat these exercises 2 to 3 times.

After 7 to 10 days, you can introduce nightly use of the GoutChew Relax home ® splint to maximize the benefits, especially if you grind your teeth (bruxism), suffer from sleep apnea, or if your snoring is annoying.

What side effects can GoutChew Relax ® cause ?

The incisors may be sensitive at the beginning of the treatment, but this disappears quickly. Hypersalivation is also normal. The salivation will gradually decrease and you will be able to wear the device continuously.

In case of gum irritation and hypersensitivity of the teeth, it is advisable to reduce the wearing time.

It may happen that the gums bleed when the mouthpiece is used for the first time. In this case, it is advisable to wear the splint gradually to ensure that the tissue strengthens and the bleeding stops spontaneously. The bioactive brace should not cause any injury or pain.

What else should we look out for ?

1. Maintenance

The bioactive mouthwash should be rinsed with warm water after each use, and brushed with a toothbrush once a day. When not in use, it should be kept in the box with the lid off for a few hours after use to allow it to dry.

2. Wear

This therapeutic device is made of silicone, which loses its elasticity and effectiveness over time, the braces have a lifespan of 3 to 6 months (depending on factors such as activity level, chewing ability, maintenance). As the benefits gradually diminish, it is recommended to replace the device after 6 months at the latest.

3. Dental care

If you are undergoing implant, prosthetic or orthodontic treatment or have periodontitis (loosening of the teeth), limit active wear to a maximum of 10 minutes per day and consult your dentist before using the cradle.

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